

**ADVIA Centaur® CP**

**Invalid Input Data Error when Scanning ADVIA Centaur CP HBsAgII Calibrator Card**

Our records indicate that your facility may have received the following product:

**Table 1. ADVIA Centaur CP Affected Product(s)**

Assay	Siemens Material Number (SMN)	Unique Device Identification (UDI)	Kit Lot	Date of Manufacture (YYYY-MM-DD)	Expiration Date (YYYY-MM-DD)
ADVIA Centaur CP HBsAgII Assay (200 Test)	10994742	(01)00630414602905(10)98885292(17)20231207	98885292	2022-12-07	2023-12-07

**Reason for Customer Notification**

The purpose of this communication is to inform you of an issue with the product indicated in Table 1 above and provide instructions on actions that your laboratory must take.

Siemens Healthcare Diagnostics Inc. has confirmed that an invalid input data error occurs when the HBsAgII Assay Kit Lot 98885292 calibrator assigned value card is scanned on the ADVIA Centaur CP system. This issue prevents customers from calibrating and testing with this assay kit lot.

Atellica IM, ADVIA Centaur XP, and ADVIA Centaur XPT are not impacted by this issue.

Siemens is currently investigating root cause.

**Actions to be Taken by the Customer**

- For the products listed in Table 1, please perform the following steps:
- Per the ADVIA Centaur CP Operator’s Guide instructions, scan the updated ADVIA Centaur CP HBsAgII Assay Kit Lot 98885292 calibrator assigned value card from Figure 1. The calibrator data will be stored to the system and allow for successful calibration to be achieved. See Figure 1 in Additional Information below for the updated calibrator card.


Please retain this letter with your laboratory records and forward this letter to those who may have received this product.

## Invalid Input Data Error when Scanning ADVIA Centaur CP HBsAgII Calibrator Card

We apologize for the inconvenience this situation may cause. If you have any questions, please contact your Siemens Healthineers Customer Care Center at 1800-815-508 or your local Siemens Healthineers technical support representative.

### Additional Information

Figure 1. ADVIA Centaur CP HBsII Assay Kit Lot 98885292 Calibrator Card (updated)

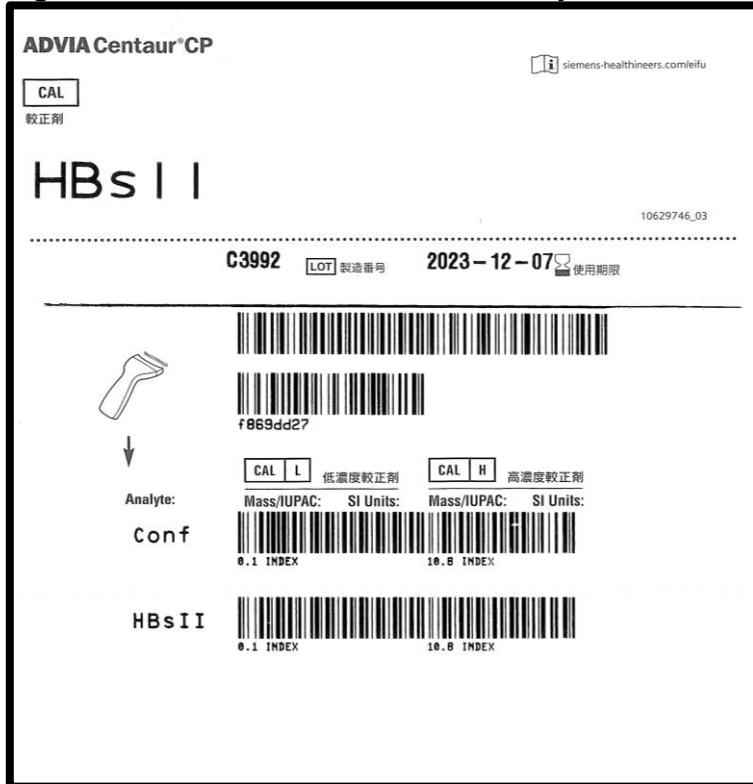

ADVIA Centaur is a trademark of Siemens Healthcare Diagnostics Inc.