

FIELD SAFETY NOTICE

MYMEDIC INNOVATION SDN BHD



No	Product	MMI Product Code	APPL Product Code
1	Ultrasound Gel (Bottle) 260g	USG90401	10.4000.01
2	Ultrasound Gel 5kg	USG90402	10.4001.01
3	ECG Gel (Bottle) 260g	ECG90403	10.0800.01
4	ECG Gel 5kg	ECG90404	10.0801.01

1. Handling Method

- a) Clean your hands and put on your gloves before using the product.
- b) Disinfect any medical equipments that is used with the product.

260g:

- i. Only use sanitized instruments to open the bottle.
- ii. To avoid contamination, dispenser tip of bottle should not come into contact with patient, staff or instruments.
- iii. Immediately close the bottle cap after using the product.
- iv. The product should be marked with the date used and then discarded after 28 days.
- v. Discard product if bottle is damaged or evidence of contamination is present.

5kg:

- i. Refilling process must be done in a clean and hygienic environment.
- ii. Reusable bottle should be cleaned with distilled water and thoroughly dried before being refilled.
- iii. Discard any excess gel before capping.
- iv. The product should be marked with the date used and then discarded after 28 days.
- v. Discard product if bottle is damaged or evidence of contamination is present.

2. Storage Method

- a) Store in a cool and dry place. (10°C - 40°C).
- b) Store away from direct sunlight.
- c) Store in a clean environment.