

Notification Letter for Distributors

Product Safety Notice **for selected HOYA Vivinex multiSert XY1-SP and XC1-SP IOLs**

30 Jun 2023

MEDFRAY MEDICAL SDN BHD
10-3-1, 3rd Floor, Queen's Avenue,
Block 10, Jalan Shelly,
55100 Kuala Lumpur, Malaysia
Tel, +603 920 11688

Dear Mr.Seiong,

This is to officially notify your organization, [Organization Name], that we are issuing a Product Safety Notice of specific lots of HOYA Vivinex multiSert IOLs (Models: XY1-SP and XC1-SP). This Product Safety Notice is limited to XY1-SP and XC1-SP models produced with specific lots from a new supplier of injectors. We are contacting you as the affected products have been supplied to your organization.

Background:

We recently concluded an initial investigation into an increase in complaints related to rapid IOL release from injectors while using Push Plunger Option to implant the lenses. In several of the complaints received, the surgeons have noticed a sudden loss of control with rapid advancement of the IOL while using Push Plunger Option to implant the lenses into the patient eye, resulting in posterior capsular rupture.

As part of our preliminary investigation, it has been confirmed that the issue does NOT affect all units of products produced with injectors of the new supplier, it is limited to specific lots and presents itself only when using Push Plunger Option for implantation of the lens into patient eye. After extensive testing of the affected injectors, we can confirm that this issue does not affect the Injector Knob Rotation Option.

We are taking all possible steps to contain the problem. We are investigating the potential root causes and are working step-by-step to implement corrective actions. Furthermore, we are committed to working closely with you to effectively and efficiently manage this situation.

This Product Safety Notice is limited to specific production lots of Vivinex multiSert IOLs (Models: XY1-SP and XC1-SP). It does not affect any other HOYA Surgical Optics IOLs that have been distributed to your organization, including Vivinex Toric, Vivinex Impress, and the Vivinex Gemetric family which are preloaded in the multiSert injector.

Action requested:


To mitigate the potential for patient harm, we have recently contacted you to block all affected stock under your control. Please refer to Appendix 1 for the complete list of impacted serial numbers.

We now kindly ask you to take the following actions immediately:

1. Confirm HOYA Surgical Optics' products received at your organization and check the Serial Numbers, which are listed on the labels, as shown in the example below, as well as in the shipping documents, against the Serial Numbers provided on Appendix 1.



2. Attach the Product Safety Notice (PSN) label below (provided to you by Hoya) to outer cartons of all Serial Numbers contained on the attached list, which have not yet been used. This includes not only the affected stock currently on hold at your warehouse, but also any consignment stock currently on hold at your customer's premises.

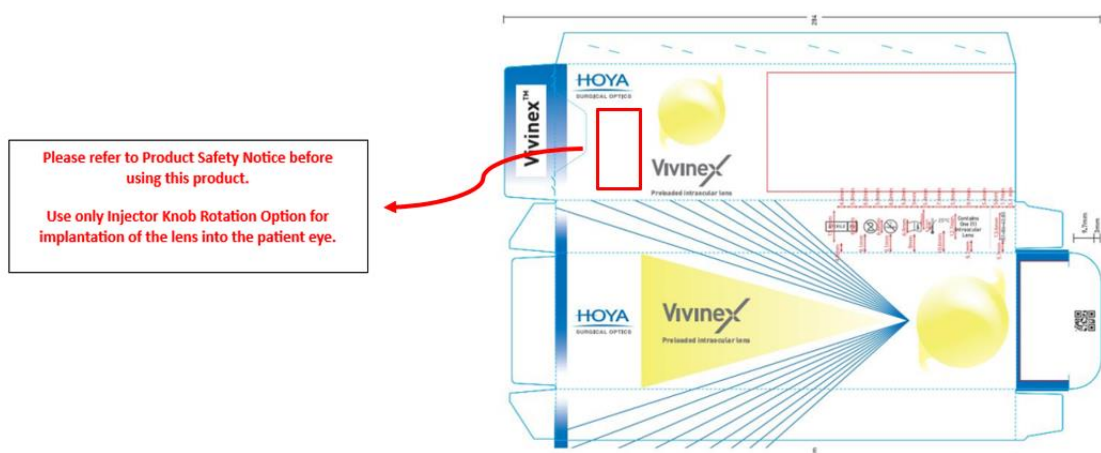

3. Attach a copy of the Product Safety Notice dated 30 June 2023 (provided to you together with this letter) to the impacted Serial Numbers before distributing the product.
4. Please complete Appendix 1- Distributor Reply Form and return it to Hoya within 15 business days.

Please note that you may still receive products affected by this Product Safety Notice in the near future, in which case these products will already be labelled with the PSN label, and you will be required only to attach a copy of the Product Safety Notice to be distributed together with the product.

This Product Safety Notice is limited to specific production lots of Vivinex multiSert IOLs (Models: XY1-SP and XC1-SP). No further action is required as it does **not** affect any other HOYA Surgical Optics IOLs that have been distributed to your organization, including Vivinex Toric, Vivinex Impress, and the Vivinex Gemetric family which are preloaded in the multiSert injector.

At HOYA Surgical Optics, our top priority is the health and safety of our patients. We apologize for any inconvenience this action may cause you and we appreciate your understanding as we act to ensure patient safety.

If you have any questions or concerns related to this matter, please contact me directly. I will be your point of contact on behalf of HOYA Surgical Optics.

Kind regards,



Ninnet Chawarangpong
QA Engineer II (APAC)

Appendix 1 - DISTRIBUTOR REPLY FORM

Organization Name: **MEDFRAY MEDICAL SDN BHD**

Section 1

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN221A2	YES	HQ	NO
XC1B-SP	2CN22104	YES	HQ	NO
XC1B-SP	2CN224Q2	YES	HQ	NO
XC1B-SP	2CN221ZA	YES	HQ	NO
XC1B-SP	2CN22206	YES	HQ	NO
XC1B-SP	2CN22392	YES	HQ	NO
XC1B-SP	2CN223P7	YES	HQ	NO
XC1B-SP	2CN2216A	YES	HQ	NO
XC1B-SP	2CN22173	YES	HQ	NO
XC1B-SP	2CN221X2	YES	HQ	NO
XY1B-SP	2DN212E1	YES	RETURNED	NO
XC1B-SP	2CN224Q5	YES	HQ	NO
XC1B-SP	2CN224R2	YES	HQ	NO
XY1B-SP	2DN215G2	YES	RETURNED	NO
XY1B-SP	2DN22RP8	YES	HQ	NO
XY1B-SP	2DN22KK8	YES	HQ	NO
XY1B-SP	2DN22KL6	YES	HQ	NO
XY1B-SP	2DN218T6	YES	HQ	NO
XC1B-SP	2CN22182	YES	HQ	NO
XC1B-SP	2CN221A9	YES	HQ	NO
XC1B-SP	2CN221Z8	YES	HQ	NO
XC1B-SP	2CN22413	YES	HQ	NO
XC1B-SP	2CN22448	YES	HQ	NO
XY1B-SP	2DN22RP3	YES	HQ	NO
XY1B-SP	2DN22KKA	YES	HQ	NO
XC1B-SP	2CN224S8	YES	HQ	NO
XC1B-SP	2CN225J2	YES	HQ	NO
XC1B-SP	2CN225F4	YES	HQ	NO
XC1B-SP	2CN225V3	YES	HQ	NO
XC1B-SP	2CN227C5	YES	HQ	NO
XC1B-SP	2CN221Z3	YES	HQ	NO
XC1B-SP	2CN22112	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22184	YES	HQ	NO
XY1B-SP	2DN218R6	YES	HQ	NO
XY1B-SP	2DN22Q26	YES	HQ	NO
XY1B-SP	2DN22QA3	YES	HQ	NO
XC1B-SP	2CN22185	YES	HQ	NO
XC1B-SP	2CN223B7	YES	HQ	NO
XC1B-SP	2CN223DA	YES	HQ	NO
XC1B-SP	2CN223N9	YES	HQ	NO
XC1B-SP	2CN2243A	YES	HQ	NO
XC1B-SP	2CN22442	YES	HQ	NO
XY1B-SP	2DN22HS7	YES	HQ	NO
XY1B-SP	2DN22HS8	YES	HQ	NO
XY1B-SP	2DN22QN5	YES	HQ	NO
XY1B-SP	2DN22RN6	YES	HQ	NO
XY1B-SP	2DN22KL4	YES	HQ	NO
XY1B-SP	2DN215FA	YES	HQ	NO
XC1B-SP	2CN22154	YES	HQ	NO
XC1B-SP	2CN2218A	YES	HQ	NO
XC1B-SP	2CN221B2	YES	HQ	NO
XC1B-SP	2CN22205	YES	HQ	NO
XC1B-SP	2CN22BC8	YES	HQ	NO
XC1B-SP	2CN22BD5	YES	HQ	NO
XC1B-SP	2CN221X6	YES	HQ	NO
XC1B-SP	2CN22202	YES	HQ	NO
XC1B-SP	2CN221A8	YES	HQ	NO
XC1B-SP	2CN225CA	YES	HQ	NO
XC1B-SP	2CN227A8	YES	HQ	NO
XC1B-SP	2CN221X9	YES	HQ	NO
XC1B-SP	2CN221XA	YES	HQ	NO
XC1B-SP	2CN223G7	YES	HQ	NO
XC1B-SP	2CN223G9	YES	HQ	NO
XC1B-SP	2CN223Z7	YES	HQ	NO
XC1B-SP	2CN223Z9	YES	HQ	NO
XC1B-SP	2CN2241A	YES	HQ	NO
XY1B-SP	2DN215J2	YES	RETURNED	NO
XC1B-SP	2CN22D82	YES	HQ	NO
XY1B-SP	2DN22P5A	YES	HQ	NO
XC1B-SP	2CN223J6	YES	HQ	NO
XC1B-SP	2CN22438	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22172	YES	HQ	NO
XC1B-SP	2CN221X7	YES	HQ	NO
XC1B-SP	2CN221Z4	YES	HQ	NO
XY1B-SP	2DN21R8A	YES	RETURNED	NO
XY1B-SP	2DN21U87	YES	RETURNED	NO
XY1B-SP	2DN21U98	YES	RETURNED	NO
XY1B-SP	2DN21U99	YES	RETURNED	NO
XC1B-SP	2CN223B9	YES	HQ	NO
XC1B-SP	2CN223C2	YES	HQ	NO
XC1B-SP	2CN223Q7	YES	HQ	NO
XY1B-SP	2DN21Z17	YES	RETURNED	NO
XY1B-SP	2DN22RP7	YES	HQ	NO
XY1B-SP	2DN215H7	YES	RETURNED	NO
XC1B-SP	2CN224R8	YES	HQ	NO
XC1B-SP	2CN224S4	YES	HQ	NO
XY1B-SP	2DN228B6	YES	RETURNED	NO
XC1B-SP	2CN2272A	YES	HQ	NO
XY1B-SP	2DN22G15	YES	HQ	NO
XC1B-SP	2CN229B1	YES	HQ	NO
XC1B-SP	2CN22C52	YES	HQ	NO
XC1B-SP	2CN22166	YES	HQ	NO
XC1B-SP	2CN22183	YES	HQ	NO
XY1B-SP	2DN21VD2	YES	RETURNED	NO
XY1B-SP	2DN21VD7	YES	RETURNED	NO
XC1B-SP	2CN222Q1	YES	HQ	NO
XC1B-SP	2CN223G1	YES	HQ	NO
XC1B-SP	2CN22404	YES	HQ	NO
XC1B-SP	2CN22443	YES	HQ	NO
XC1B-SP	2CN22446	YES	HQ	NO
XC1B-SP	2CN22133	YES	HQ	NO
XC1B-SP	2CN22175	YES	HQ	NO
XC1B-SP	2CN2217A	YES	HQ	NO
XC1B-SP	2CN221C4	YES	HQ	NO
XC1B-SP	2CN221Y2	YES	HQ	NO
XY1B-SP	2DN21WB8	YES	RETURNED	NO
XY1B-SP	2DN21XK8	YES	RETURNED	NO
XY1B-SP	2DN215C6	YES	RETURNED	NO
XY1B-SP	2DN215H4	YES	USED	N/A

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN215H5	YES	USED	N/A
XY1B-SP	2DN215H9	YES	RETURNED	NO
XY1B-SP	2DN21TC9	YES	RETURNED	NO
XY1B-SP	2DN21U93	YES	RETURNED	NO
XY1B-SP	2DN21U94	YES	RETURNED	NO
XC1B-SP	2CN225R7	YES	HQ	NO
XC1B-SP	2CN22724	YES	HQ	NO
XC1B-SP	2CN222P1	YES	HQ	NO
XC1B-SP	2CN222U9	YES	HQ	NO
XC1B-SP	2CN223B8	YES	HQ	NO
XC1B-SP	2CN223C1	YES	HQ	NO
XC1B-SP	2CN22412	YES	HQ	NO
XY1B-SP	2DN227Q2	YES	RETURNED	NO
XY1B-SP	2DN22AK8	YES	RETURNED	NO
XY1B-SP	2DN22AKA	YES	RETURNED	NO
XC1B-SP	2CN22668	YES	HQ	NO
XC1B-SP	2CN2268A	YES	HQ	NO
XC1B-SP	2CN22739	YES	HQ	NO
XY1B-SP	2DN215H8	YES	RETURNED	NO
XY1B-SP	2DN215J1	YES	RETURNED	NO
XC1B-SP	2CN2210A	YES	HQ	NO
XC1B-SP	2CN22163	YES	HQ	NO
XC1B-SP	2CN221A6	YES	HQ	NO
XY1B-SP	2DN21Z07	YES	RETURNED	NO
XY1B-SP	2DN21Z44	YES	RETURNED	NO
XY1B-SP	2DN21Z45	YES	RETURNED	NO
XY1B-SP	2DN21HU7	YES	RETURNED	NO
XY1B-SP	2DN21HU9	YES	RETURNED	NO
XC1B-SP	2CN228R8	YES	HQ	NO
XY1B-SP	2DN215C3	YES	RETURNED	NO
XC1B-SP	2CN22356	YES	HQ	NO
XC1B-SP	2CN22BCA	YES	HQ	NO
XY1B-SP	2DN215B6	YES	RETURNED	NO
XY1B-SP	2DN215D2	YES	RETURNED	NO
XY1B-SP	2DN215JA	YES	RETURNED	NO
XC1B-SP	2CN224Q4	YES	HQ	NO
XC1B-SP	2CN224S5	YES	HQ	NO
XC1B-SP	2CN22664	YES	HQ	NO
XC1B-SP	2CN22164	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN221Y7	YES	HQ	NO
XC1B-SP	2CN221Y9	YES	HQ	NO
XC1B-SP	2CN223G2	YES	HQ	NO
XC1B-SP	2CN224S3	YES	HQ	NO
XC1B-SP	2CN226J8	YES	HQ	NO
XC1B-SP	2CN227E6	YES	HQ	NO
XC1B-SP	2CN221X8	YES	HQ	NO
XC1B-SP	2CN22201	YES	HQ	NO
XC1B-SP	2CN22203	YES	HQ	NO
XC1B-SP	2CN224Q9	YES	HQ	NO
XC1B-SP	2CN225E5	YES	HQ	NO
XC1B-SP	2CN226B7	YES	HQ	NO
XC1B-SP	2CN22757	YES	HQ	NO
XC1B-SP	2CN226H1	YES	HQ	NO
XY1B-SP	2DN21VD1	YES	RETURNED	NO
XY1B-SP	2DN21L66	YES	RETURNED	NO
XY1B-SP	2DN21HT1	YES	RETURNED	NO
XY1B-SP	2DN21HT2	YES	RETURNED	NO
XY1B-SP	2DN21S47	YES	RETURNED	NO
XY1B-SP	2DN21S48	YES	RETURNED	NO
XY1B-SP	2DN21S49	YES	RETURNED	NO
XY1B-SP	2DN21Y25	YES	RETURNED	NO
XY1B-SP	2DN21Z23	YES	RETURNED	NO
XC1B-SP	2CN226F3	YES	HQ	NO
XY1B-SP	2DN21U8A	YES	RETURNED	NO
XY1B-SP	2DN21XK3	YES	RETURNED	NO
XY1B-SP	2DN21Y27	YES	RETURNED	NO
XY1B-SP	2DN21Z12	YES	RETURNED	NO
XY1B-SP	2DN21Z19	YES	RETURNED	NO
XC1B-SP	2CN22101	YES	HQ	NO
XC1B-SP	2CN22167	YES	HQ	NO
XC1B-SP	2CN22174	YES	HQ	NO
XC1B-SP	2CN221Y1	YES	HQ	NO
XC1B-SP	2CN223D9	YES	HQ	NO
XC1B-SP	2CN223P3	YES	HQ	NO
XC1B-SP	2CN22407	YES	HQ	NO
XC1B-SP	2CN22416	YES	HQ	NO
XY1B-SP	2DN22PY7	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN22KK4	YES	HQ	NO
XC1B-SP	2CN22191	YES	HQ	NO
XC1B-SP	2CN221X3	YES	HQ	NO
XC1B-SP	2CN22207	YES	HQ	NO
XY1B-SP	2DN22P41	YES	HQ	NO
XY1B-SP	2DN22QW3	YES	HQ	NO
XC1B-SP	2CN223C4	YES	HQ	NO
XC1B-SP	2CN223C5	YES	HQ	NO
XC1B-SP	2CN223E1	YES	HQ	NO
XC1B-SP	2CN223G6	YES	HQ	NO
XC1B-SP	2CN223Z6	YES	HQ	NO
XC1B-SP	2CN22422	YES	HQ	NO
XY1B-SP	2DN22RN1	YES	HQ	NO
XY1B-SP	2DN22RP5	YES	HQ	NO
XY1B-SP	2DN215J3	YES	RETURNED	NO
XY1B-SP	2DN215J8	YES	RETURNED	NO
XC1B-SP	2CN225V7	YES	HQ	NO
XC1B-SP	2CN22DF2	YES	HQ	NO
XY1B-SP	2DN22P63	YES	HQ	NO
XY1B-SP	2DN22P64	YES	HQ	NO
XY1B-SP	2DN215D3	YES	RETURNED	NO
XY1B-SP	2DN215GA	YES	USED	N/A
XY1B-SP	2DN215H6	YES	HQ	NO
XC1B-SP	2CN22444	YES	HQ	NO
XY1B-SP	2DN215D1	YES	RETURNED	NO
XC1B-SP	2CN227D6	YES	HQ	NO
XC1B-SP	2CN227W8	YES	HQ	NO
XY1B-SP	2DN21R92	YES	RETURNED	NO
XY1B-SP	2DN22891	YES	RETURNED	NO
XY1B-SP	2DN21TC8	YES	RETURNED	NO
XY1B-SP	2DN21U88	YES	RETURNED	NO
XY1B-SP	2DN22AM1	YES	RETURNED	NO
XY1B-SP	2DN22AM2	YES	RETURNED	NO
XY1B-SP	2DN22AZA	YES	RETURNED	NO
XY1B-SP	2DN21WB5	YES	RETURNED	NO
XY1B-SP	2DN21WM9	YES	RETURNED	NO
XY1B-SP	2DN21HT8	YES	RETURNED	NO
XY1B-SP	2DN21XD3	YES	RETURNED	NO
XY1B-SP	2DN21R93	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN21VK6	YES	RETURNED	NO
XY1B-SP	2DN21U96	YES	RETURNED	NO
XC1B-SP	2CN225Y1	YES	HQ	NO
XC1B-SP	2CN228T7	YES	HQ	NO
XC1B-SP	2CN228T9	YES	HQ	NO
XC1B-SP	2CN229F3	YES	HQ	NO
XC1B-SP	2CN22AG9	YES	HQ	NO
XY1B-SP	2DN21WB6	YES	RETURNED	NO
XY1B-SP	2DN21Y23	YES	RETURNED	NO
XY1B-SP	2DN21HT4	YES	RETURNED	NO
XY1B-SP	2DN21HT5	YES	RETURNED	NO
XC1B-SP	2CN221X5	YES	HQ	NO
XC1B-SP	2CN221Z7	YES	HQ	NO
XY1B-SP	2DN22HS3	YES	HQ	NO
XY1B-SP	2DN22HS4	YES	HQ	NO
XY1B-SP	2DN21U91	YES	RETURNED	NO
XY1B-SP	2DN21UA5	YES	RETURNED	NO
XY1B-SP	2DN21VK9	YES	RETURNED	NO
XC1B-SP	2CN229H6	YES	HQ	NO
XC1B-SP	2CN229J7	YES	HQ	NO
XC1B-SP	2CN22BD6	YES	HQ	NO
XY1B-SP	2DN21PM4	YES	RETURNED	NO
XY1B-SP	2DN21Y22	YES	RETURNED	NO
XC1B-SP	2CN22161	YES	HQ	NO
XC1B-SP	2CN22189	YES	HQ	NO
XC1B-SP	2CN221Y6	YES	HQ	NO
XY1B-SP	2DN21LB6	YES	RETURNED	NO
XY1B-SP	2DN21VD6	YES	RETURNED	NO
XY1B-SP	2DN21WN1	YES	RETURNED	NO
XY1B-SP	2DN227N8	YES	RETURNED	NO
XY1B-SP	2DN227R6	YES	RETURNED	NO
XC1B-SP	2CN22BD4	YES	HQ	NO
XC1B-SP	2CN22C26	YES	HQ	NO
XC1B-SP	2CN22186	YES	HQ	NO
XC1B-SP	2CN221Y4	YES	HQ	NO
XC1B-SP	2CN223FA	YES	HQ	NO
XY1B-SP	2DN21QJ5	YES	RETURNED	NO
XY1B-SP	2DN215B7	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN215B9	YES	RETURNED	NO
XY1B-SP	2DN215C4	YES	RETURNED	NO
XY1B-SP	2DN215C5	YES	RETURNED	NO
XY1B-SP	2DN215H2	YES	HQ	NO
XC1B-SP	2CN226H8	YES	HQ	NO
XC1B-SP	2CN22717	YES	HQ	NO
XY1B-SP	2DN22874	YES	RETURNED	NO
XC1B-SP	2CN223P2	YES	HQ	NO
XC1B-SP	2CN2242A	YES	HQ	NO
XC1B-SP	2CN221Z6	YES	HQ	NO
XC1B-SP	2CN223PA	YES	HQ	NO
XC1B-SP	2CN22408	YES	HQ	NO
XY1B-SP	2DN21Z31	YES	RETURNED	NO
XY1B-SP	2DN21Z32	YES	RETURNED	NO
XY1B-SP	2DN21Z38	YES	RETURNED	NO
XY1B-SP	2DN21Z39	YES	RETURNED	NO
XY1B-SP	2DN22AL6	YES	RETURNED	NO
XY1B-SP	2DN22AMA	YES	RETURNED	NO
XY1B-SP	2DN22AZ6	YES	RETURNED	NO
XY1B-SP	2DN215C1	YES	RETURNED	NO
XY1B-SP	2DN215D4	YES	RETURNED	NO
XY1B-SP	2DN215D5	YES	RETURNED	NO
XY1B-SP	2DN215F3	YES	USED	N/A
XY1B-SP	2DN215F8	YES	RETURNED	NO
XC1B-SP	2CN224Q7	YES	HQ	NO
XC1B-SP	2CN227E1	YES	HQ	NO
XC1B-SP	2CN227EA	YES	HQ	NO
XY1B-SP	2DN21HT3	YES	RETURNED	NO
XC1B-SP	2CN228V9	YES	HQ	NO
XC1B-SP	2CN2299A	YES	HQ	NO
XC1B-SP	2CN229B2	YES	HQ	NO
XC1B-SP	2CN22AG2	YES	HQ	NO
XY1B-SP	2DN21VPA	YES	RETURNED	NO
XY1B-SP	2DN21WC1	YES	RETURNED	NO
XY1B-SP	2DN22P65	YES	HQ	NO
XC1B-SP	2CN227B9	YES	HQ	NO
XY1B-SP	2DN22RP2	YES	HQ	NO
XY1B-SP	2DN22KL7	YES	HQ	NO
XC1B-SP	2CN22DN2	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN22P3A	YES	HQ	NO
XY1B-SP	2DN22P72	YES	HQ	NO
XY1B-SP	2DN21R91	YES	RETURNED	NO
XY1B-SP	2DN21VK3	YES	RETURNED	NO
XY1B-SP	2DN21XL1	YES	RETURNED	NO
XY1B-SP	2DN21Z14	YES	RETURNED	NO
XY1B-SP	2DN22877	YES	RETURNED	NO
XY1B-SP	2DN22878	YES	RETURNED	NO
XY1B-SP	2DN228X4	YES	RETURNED	NO
XY1B-SP	2DN228X5	YES	RETURNED	NO
XY1B-SP	2DN22AL5	YES	RETURNED	NO
XY1B-SP	2DN22AM4	YES	RETURNED	NO
XY1B-SP	2DN22AM5	YES	RETURNED	NO
XY1B-SP	2DN22AN8	YES	RETURNED	NO
XY1B-SP	2DN22B12	YES	RETURNED	NO
XY1B-SP	2DN21Z15	YES	RETURNED	NO
XY1B-SP	2DN21Z33	YES	RETURNED	NO
XY1B-SP	2DN21Z37	YES	RETURNED	NO
XY1B-SP	2DN21Z3A	YES	RETURNED	NO
XY1B-SP	2DN22AL2	YES	RETURNED	NO
XY1B-SP	2DN22AL3	YES	RETURNED	NO
XY1B-SP	2DN22AN4	YES	RETURNED	NO
XY1B-SP	2DN22AN6	YES	RETURNED	NO
XC1B-SP	2CN22C91	YES	HQ	NO
XY1B-SP	2DN21Z41	YES	RETURNED	NO
XY1B-SP	2DN21Z42	YES	RETURNED	NO
XY1B-SP	2DN21U85	YES	RETURNED	NO
XY1B-SP	2DN21WC2	YES	RETURNED	NO
XY1B-SP	2DN22AZ4	YES	RETURNED	NO
XY1B-SP	2DN22AZ5	YES	RETURNED	NO
XY1B-SP	2DN21Z0A	YES	RETURNED	NO
XY1B-SP	2DN215B4	YES	RETURNED	NO
XY1B-SP	2DN215B5	YES	RETURNED	NO
XY1B-SP	2DN215F6	YES	HQ	NO
XY1B-SP	2DN215J7	YES	RETURNED	NO
XY1B-SP	2DN215K4	YES	RETURNED	NO
XY1B-SP	2DN215K5	YES	RETURNED	NO
XC1B-SP	2CN224PA	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN2285A	YES	RETURNED	NO
XC1B-SP	2CN227Z3	YES	HQ	NO
XC1B-SP	2CN2293A	YES	HQ	NO
XY1B-SP	2DN22QL9	YES	HQ	NO
XC1B-SP	2CN22CJ2	YES	HQ	NO
XY1B-SP	2DN21WN2	YES	RETURNED	NO
XY1B-SP	2DN21Z08	YES	RETURNED	NO
XY1B-SP	2DN21Z09	YES	RETURNED	NO
XY1B-SP	2DN21Z34	YES	RETURNED	NO
XY1B-SP	2DN21Z35	YES	RETURNED	NO
XY1B-SP	2DN21VK1	YES	RETURNED	NO
XY1B-SP	2DN21Z43	YES	RETURNED	NO
XY1B-SP	2DN21VCA	YES	RETURNED	NO
XY1B-SP	2DN21VK4	YES	RETURNED	NO
XY1B-SP	2DN21VK5	YES	RETURNED	NO
XY1B-SP	2DN225U2	YES	HQ	NO
XY1B-SP	2DN22AN7	YES	RETURNED	NO
XY1B-SP	2DN21R84	YES	RETURNED	NO
XY1B-SP	2DN21S46	YES	RETURNED	NO
XC1B-SP	2CN22AG7	YES	HQ	NO
XC1B-SP	2CN22AX1	YES	HQ	NO
XY1B-SP	2DN21VD3	YES	RETURNED	NO
XY1B-SP	2DN21VD4	YES	RETURNED	NO
XY1B-SP	2DN21VD5	YES	RETURNED	NO
XY1B-SP	2DN21Z11	YES	RETURNED	NO
XY1B-SP	2DN21Z18	YES	RETURNED	NO
XY1B-SP	2DN21Z1A	YES	RETURNED	NO
XY1B-SP	2DN22514	YES	HQ	NO
XY1B-SP	2DN22515	YES	HQ	NO
XC1B-SP	2CN22749	YES	HQ	NO
XY1B-SP	2DN21R94	YES	RETURNED	NO
XY1B-SP	2DN227N4	YES	RETURNED	NO
XY1B-SP	2DN22881	YES	RETURNED	NO
XY1B-SP	2DN21U95	YES	RETURNED	NO
XY1B-SP	2DN21XL5	YES	RETURNED	NO
XY1B-SP	2DN21TC7	YES	RETURNED	NO
XY1B-SP	2DN21VK7	YES	RETURNED	NO
XY1B-SP	2DN21VK8	YES	RETURNED	NO
XC1B-SP	2CN22BD2	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22162	YES	HQ	NO
XC1B-SP	2CN221A7	YES	HQ	NO
XC1B-SP	2CN221C7	YES	HQ	NO
XC1B-SP	2CN223Y2	YES	HQ	NO
XC1B-SP	2CN2244A	YES	HQ	NO
XY1B-SP	2DN21WB7	YES	RETURNED	NO
XY1B-SP	2DN21XZ6	YES	RETURNED	NO
XY1B-SP	2DN215B8	YES	RETURNED	NO
XY1B-SP	2DN215K3	YES	RETURNED	NO
XC1B-SP	2CN223Y3	YES	HQ	NO
XY1B-SP	2DN22RN7	YES	HQ	NO
XY1B-SP	2DN22P59	YES	HQ	NO
XY1B-SP	2DN22RN8	YES	HQ	NO
XY1B-SP	2DN21UA1	YES	RETURNED	NO
XY1B-SP	2DN21XK5	YES	RETURNED	NO
XY1B-SP	2DN215CA	YES	RETURNED	NO
XC1B-SP	2CN224Q3	YES	HQ	NO
XC1B-SP	2CN224Q6	YES	HQ	NO
XC1B-SP	2CN22669	YES	HQ	NO
XC1B-SP	2CN226J4	YES	HQ	NO
XY1B-SP	2DN21282	YES	RETURNED	NO
XC1B-SP	2CN22156	YES	HQ	NO
XC1B-SP	2CN222U4	YES	HQ	NO
XC1B-SP	2CN223B5	YES	HQ	NO
XC1B-SP	2CN223F9	YES	HQ	NO
XC1B-SP	2CN223GA	YES	HQ	NO
XC1B-SP	2CN223ZA	YES	HQ	NO
XC1B-SP	2CN22401	YES	HQ	NO
XC1B-SP	2CN22405	YES	HQ	NO
XC1B-SP	2CN22419	YES	HQ	NO
XC1B-SP	2CN22449	YES	HQ	NO
XY1B-SP	2DN218M3	YES	HQ	NO
XY1B-SP	2DN215D6	YES	RETURNED	NO
XC1B-SP	2CN224R3	YES	HQ	NO
XC1B-SP	2CN224S2	YES	HQ	NO
XY1B-SP	2DN225U3	YES	HQ	NO
XY1B-SP	2DN22P32	YES	HQ	NO
XY1B-SP	2DN22RN5	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22DN8	YES	HQ	NO
XC1B-SP	2CN22DX1	YES	HQ	NO
XY1B-SP	2DN22RNA	YES	HQ	NO
XY1B-SP	2DN22KK6	YES	HQ	NO
XC1B-SP	2CN22DW2	YES	HQ	NO
XY1B-SP	2DN22QZ8	YES	HQ	NO
XC1B-SP	2CN22DW6	YES	HQ	NO
XY1B-SP	2DN22P37	YES	HQ	NO
XY1B-SP	2DN22P42	YES	USED	N/A
XY1B-SP	2DN22P68	YES	USED	N/A
XY1B-SP	2DN22RN3	YES	HQ	NO
XY1B-SP	2DN22KK9	YES	HQ	NO
XY1B-SP	2DN22P23	YES	HQ	NO
XY1B-SP	2DN22P24	YES	HQ	NO
XY1B-SP	2DN22P25	YES	HQ	NO
XY1B-SP	2DN22P56	YES	HQ	NO
XY1B-SP	2DN22P57	YES	HQ	NO
XC1B-SP	2CN22DN3	YES	HQ	NO
XC1B-SP	2CN22DN4	YES	HQ	NO
XC1B-SP	2CN22DWA	YES	HQ	NO
XC1B-SP	2CN22DN1	YES	HQ	NO
XY1B-SP	2DN22RP6	YES	HQ	NO
XY1B-SP	2DN22P2A	YES	HQ	NO
XY1B-SP	2DN22P44	YES	USED	N/A
XY1B-SP	2DN22P45	YES	HQ	NO
XY1B-SP	2DN22P69	YES	HQ	NO
XY1B-SP	2DN22QU1	YES	HQ	NO
XY1B-SP	2DN22RN2	YES	HQ	NO
XY1B-SP	2DN22RN9	YES	HQ	NO
XY1B-SP	2DN22KL2	YES	HQ	NO
XY1B-SP	2DN22P39	YES	HQ	NO
XY1B-SP	2DN22RP1	YES	HQ	NO
XY1B-SP	2DN22RP4	YES	HQ	NO
XY1B-SP	2DN22KK7	YES	HQ	NO
XY1B-SP	2DN22P22	YES	HQ	NO
XY1B-SP	2DN22P49	YES	HQ	NO
XY1B-SP	2DN22P58	YES	HQ	NO
XY1B-SP	2DN22QS1	YES	HQ	NO
XC1B-SP	2CN22DN5	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22DN6	YES	HQ	NO
XC1B-SP	2CN22DN7	YES	HQ	NO
XY1B-SP	2DN22P43	YES	USED	N/A
XC1B-SP	2CN22DW8	YES	HQ	NO
XC1B-SP	2CN22D98	YES	HQ	NO
XC1B-SP	2CN22D77	YES	HQ	NO
XC1B-SP	2CN22D95	YES	HQ	NO
XC1B-SP	2CN22DA4	YES	HQ	NO
XC1B-SP	2CN22DF1	YES	HQ	NO
XY1B-SP	2DN22KK2	YES	HQ	NO
XY1B-SP	2DN22P48	YES	HQ	NO
XY1B-SP	2DN22P33	YES	HQ	NO
XY1B-SP	2DN22P34	YES	HQ	NO
XY1B-SP	2DN22P35	YES	HQ	NO
XY1B-SP	2DN22P4A	YES	HQ	NO
XY1B-SP	2DN22P74	YES	HQ	NO
XY1B-SP	2DN22P75	YES	HQ	NO
XY1B-SP	2DN22P76	YES	HQ	NO
XC1B-SP	2CN22DW3	YES	HQ	NO
XC1B-SP	2CN22DW4	YES	HQ	NO
XY1B-SP	2DN22P29	YES	HQ	NO
XY1B-SP	2DN22P36	YES	HQ	NO
XY1B-SP	2DN22P73	YES	HQ	NO
XC1B-SP	2CN22DW7	YES	HQ	NO
XC1B-SP	2CN22DW9	YES	HQ	NO
XY1B-SP	2DN22P46	YES	USED	N/A
XY1B-SP	2DN22P47	YES	HQ	NO
XY1B-SP	2DN22P52	YES	HQ	NO
XY1B-SP	2DN22P71	YES	HQ	NO
XY1B-SP	2DN22P61	YES	HQ	NO
XC1B-SP	2CN22165	YES	HQ	NO
XC1B-SP	2CN22177	YES	HQ	NO
XC1B-SP	2CN221C6	YES	HQ	NO
XC1B-SP	2CN221Y8	YES	HQ	NO
XC1B-SP	2CN223B6	YES	HQ	NO
XC1B-SP	2CN223G8	YES	HQ	NO
XC1B-SP	2CN223P4	YES	HQ	NO
XY1B-SP	2DN22RPA	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN22QW4	YES	HQ	NO
XY1B-SP	2DN215F1	YES	HQ	NO
XY1B-SP	2DN215G7	YES	HQ	NO
XY1B-SP	2DN215J6	YES	RETURNED	NO
XC1B-SP	2CN224R4	YES	HQ	NO
XC1B-SP	2CN224S1	YES	HQ	NO
XC1B-SP	2CN225H3	YES	HQ	NO
XY1B-SP	2DN22P28	YES	HQ	NO
XC1B-SP	2CN221A3	YES	HQ	NO
XC1B-SP	2CN221B1	YES	HQ	NO
XC1B-SP	2CN221C5	YES	HQ	NO
XC1B-SP	2CN222X1	YES	HQ	NO
XC1B-SP	2CN223Y5	YES	HQ	NO
XC1B-SP	2CN22429	YES	HQ	NO
XY1B-SP	2DN22P26	YES	HQ	NO
XY1B-SP	2DN22P27	YES	HQ	NO
XY1B-SP	2DN22P53	YES	HQ	NO
XY1B-SP	2DN22P54	YES	HQ	NO
XY1B-SP	2DN22P55	YES	HQ	NO
XY1B-SP	2DN215F4	YES	HQ	NO
XY1B-SP	2DN215F5	YES	HQ	NO
XY1B-SP	2DN215F9	YES	RETURNED	NO
XY1B-SP	2DN215G6	YES	USED	N/A
XY1B-SP	2DN215HA	YES	RETURNED	NO
XY1B-SP	2DN215K2	YES	RETURNED	NO
XC1B-SP	2CN224Q8	YES	HQ	NO
XC1B-SP	2CN224S6	YES	HQ	NO
XY1B-SP	2DN22P38	YES	HQ	NO
XY1B-SP	2DN22P67	YES	HQ	NO
XY1B-SP	2DN21HT9	YES	RETURNED	NO
XY1B-SP	2DN22HS6	YES	HQ	NO
XY1B-SP	2DN22HT3	YES	HQ	NO
XC1B-SP	2CN22DD8	YES	HQ	NO
XY1B-SP	2DN22P66	YES	USED	N/A
XY1B-SP	2DN21R98	YES	RETURNED	NO
XY1B-SP	2DN22876	YES	RETURNED	NO
XY1B-SP	2DN22883	YES	RETURNED	NO
XY1B-SP	2DN22884	YES	RETURNED	NO
XY1B-SP	2DN22885	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN21Z16	YES	RETURNED	NO
XC1B-SP	2CN22102	YES	HQ	NO
XC1B-SP	2CN22158	YES	HQ	NO
XC1B-SP	2CN22179	YES	HQ	NO
XC1B-SP	2CN221Z2	YES	HQ	NO
XC1B-SP	2CN222S5	YES	HQ	NO
XC1B-SP	2CN223F8	YES	HQ	NO
XC1B-SP	2CN22418	YES	HQ	NO
XY1B-SP	2DN225U1	YES	HQ	NO
XY1B-SP	2DN227Q3	YES	RETURNED	NO
XY1B-SP	2DN22KL5	YES	HQ	NO
XY1B-SP	2DN22AZ3	YES	RETURNED	NO
XY1B-SP	2DN22HSA	YES	HQ	NO
XC1B-SP	2CN22A86	YES	HQ	NO
XC1B-SP	2CN22AFA	YES	HQ	NO
XC1B-SP	2CN22111	YES	HQ	NO
XC1B-SP	2CN22134	YES	HQ	NO
XC1B-SP	2CN22157	YES	HQ	NO
XC1B-SP	2CN2215A	YES	HQ	NO
XC1B-SP	2CN221YA	YES	HQ	NO
XC1B-SP	2CN221Z1	YES	HQ	NO
XC1B-SP	2CN22DX5	YES	HQ	NO
XC1B-SP	2CN22DX6	YES	HQ	NO
XC1B-SP	2CN22DX7	YES	HQ	NO
XC1B-SP	2CN22417	YES	HQ	NO
XC1B-SP	2CN22423	YES	HQ	NO
XC1B-SP	2CN22426	YES	HQ	NO
XY1B-SP	2DN215G3	YES	USED	N/A
XY1B-SP	2DN215G8	YES	HQ	NO
XC1B-SP	2CN224R6	YES	HQ	NO
XC1B-SP	2CN226DA	YES	HQ	NO
XY1B-SP	2DN215C8	YES	RETURNED	NO
XY1B-SP	2DN215G4	YES	RETURNED	NO
XY1B-SP	2DN215G5	YES	HQ	NO
XY1B-SP	2DN215G9	YES	HQ	NO
XY1B-SP	2DN22887	YES	RETURNED	NO
XY1B-SP	2DN22888	YES	RETURNED	NO
XY1B-SP	2DN21HU4	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22153	YES	HQ	NO
XC1B-SP	2CN22192	YES	HQ	NO
XC1B-SP	2CN221A1	YES	HQ	NO
XC1B-SP	2CN221A5	YES	HQ	NO
XC1B-SP	2CN221X1	YES	HQ	NO
XC1B-SP	2CN221Y3	YES	HQ	NO
XC1B-SP	2CN221Y5	YES	HQ	NO
XC1B-SP	2CN221Z9	YES	HQ	NO
XC1B-SP	2CN22103	YES	HQ	NO
XC1B-SP	2CN22105	YES	HQ	NO
XC1B-SP	2CN22151	YES	HQ	NO
XC1B-SP	2CN223S8	YES	HQ	NO
XY1B-SP	2DN21R86	YES	RETURNED	NO
XY1B-SP	2DN21U97	YES	RETURNED	NO
XY1B-SP	2DN21U9A	YES	RETURNED	NO
XY1B-SP	2DN21UA7	YES	RETURNED	NO
XC1B-SP	2CN22414	YES	HQ	NO
XC1B-SP	2CN22178	YES	HQ	NO
XC1B-SP	2CN221X4	YES	HQ	NO
XC1B-SP	2CN22208	YES	HQ	NO
XY1B-SP	2DN21VK2	YES	RETURNED	NO
XY1B-SP	2DN21XE1	YES	RETURNED	NO
XY1B-SP	2DN21XK4	YES	RETURNED	NO
XY1B-SP	2DN21XK6	YES	RETURNED	NO
XY1B-SP	2DN21XL3	YES	RETURNED	NO
XC1B-SP	2CN222Q5	YES	HQ	NO
XC1B-SP	2CN223G4	YES	HQ	NO
XC1B-SP	2CN223G5	YES	HQ	NO
XC1B-SP	2CN223N5	YES	HQ	NO
XC1B-SP	2CN22421	YES	HQ	NO
XC1B-SP	2CN22425	YES	HQ	NO
XC1B-SP	2CN22437	YES	HQ	NO
XY1B-SP	2DN22AM6	YES	RETURNED	NO
XY1B-SP	2DN22AZ8	YES	RETURNED	NO
XY1B-SP	2DN22HS5	YES	HQ	NO
XY1B-SP	2DN227P1	YES	RETURNED	NO
XY1B-SP	2DN22873	YES	RETURNED	NO
XY1B-SP	2DN2287A	YES	RETURNED	NO
XY1B-SP	2DN215BA	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN215C7	YES	RETURNED	NO
XY1B-SP	2DN215C9	YES	RETURNED	NO
XC1B-SP	2CN224R1	YES	HQ	NO
XY1B-SP	2DN22B13	YES	RETURNED	NO
XC1B-SP	2CN22738	YES	HQ	NO
XC1B-SP	2CN227F2	YES	HQ	NO
XY1B-SP	2DN22B01	YES	RETURNED	NO
XC1B-SP	2CN22955	YES	HQ	NO
XC1B-SP	2CN22A94	YES	HQ	NO
XY1B-SP	2DN227P3	YES	RETURNED	NO
XY1B-SP	2DN22879	YES	RETURNED	NO
XY1B-SP	2DN22B05	YES	RETURNED	NO
XC1B-SP	2CN227Y6	YES	HQ	NO
XC1B-SP	2CN22AY8	YES	HQ	NO
XC1B-SP	2CN22C98	YES	HQ	NO
XY1B-SP	2DN22AM7	YES	RETURNED	NO
XY1B-SP	2DN22AN1	YES	RETURNED	NO
XY1B-SP	2DN21WM7	YES	RETURNED	NO
XY1B-SP	2DN21HU5	YES	RETURNED	NO
XY1B-SP	2DN22B0A	YES	RETURNED	NO
XC1B-SP	2CN22427	YES	HQ	NO
XY1B-SP	2DN215G1	YES	HQ	NO
XY1B-SP	2DN215J4	YES	RETURNED	NO
XY1B-SP	2DN215J5	YES	HQ	NO
XY1B-SP	2DN215J9	YES	RETURNED	NO
XC1B-SP	2CN224S7	YES	HQ	NO
XC1B-SP	2CN225F6	YES	HQ	NO
XY1B-SP	2DN21WMA	YES	RETURNED	NO
XC1B-SP	2CN22181	YES	HQ	NO
XC1B-SP	2CN221A4	YES	HQ	NO
XC1B-SP	2CN22BD1	YES	HQ	NO
XC1B-SP	2CN223G3	YES	HQ	NO
XC1B-SP	2CN223M4	YES	HQ	NO
XC1B-SP	2CN22C7A	YES	HQ	NO
XC1B-SP	2CN22C81	YES	HQ	NO
XY1B-SP	2DN21XK7	YES	RETURNED	NO
XY1B-SP	2DN21Y28	YES	RETURNED	NO
XY1B-SP	2DN21Z21	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN21TCA	YES	RETURNED	NO
XC1B-SP	2CN22A92	YES	HQ	NO
XC1B-SP	2CN22CD6	YES	HQ	NO
XC1B-SP	2CN22CG2	YES	HQ	NO
XY1B-SP	2DN22AL7	YES	RETURNED	NO
XC1B-SP	2CN22935	YES	HQ	NO
XC1B-SP	2CN22944	YES	HQ	NO
XY1B-SP	2DN218FA	YES	HQ	NO
XC1B-SP	2CN22A82	YES	HQ	NO
XC1B-SP	2CN22AX4	YES	HQ	NO
XC1B-SP	2CN22BC9	YES	HQ	NO
XY1B-SP	2DN215H3	YES	USED	N/A
XY1B-SP	2DN21HU6	YES	RETURNED	NO
XY1B-SP	2DN21R89	YES	RETURNED	NO
XY1B-SP	2DN21VP9	YES	RETURNED	NO
XY1B-SP	2DN21R96	YES	RETURNED	NO
XY1B-SP	2DN21U86	YES	RETURNED	NO
XY1B-SP	2DN21XK2	YES	RETURNED	NO
XY1B-SP	2DN21XL2	YES	RETURNED	NO
XY1B-SP	2DN21Z36	YES	RETURNED	NO
XY1B-SP	2DN21XL4	YES	RETURNED	NO
XY1B-SP	2DN22886	YES	RETURNED	NO
XY1B-SP	2DN22AZ9	YES	RETURNED	NO
XC1B-SP	2CN227E7	YES	HQ	NO
XY1B-SP	2DN21UA3	YES	RETURNED	NO
XC1B-SP	2CN227Y7	YES	HQ	NO
XY1B-SP	2DN22892	YES	RETURNED	NO
XY1B-SP	2DN21Y24	YES	RETURNED	NO
XY1B-SP	2DN22HS1	YES	HQ	NO
XY1B-SP	2DN22HS2	YES	HQ	NO
XY1B-SP	2DN21UA2	YES	RETURNED	NO
XY1B-SP	2DN21Y26	YES	RETURNED	NO
XY1B-SP	2DN21PM3	YES	RETURNED	NO
XY1B-SP	2DN2288A	YES	RETURNED	NO
XY1B-SP	2DN21U92	YES	RETURNED	NO
XY1B-SP	2DN22AK9	YES	RETURNED	NO
XY1B-SP	2DN21HT6	YES	RETURNED	NO
XC1B-SP	2CN22906	YES	HQ	NO
XC1B-SP	2CN22A85	YES	HQ	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XC1B-SP	2CN22A91	YES	HQ	NO
XC1B-SP	2CN22C67	YES	HQ	NO
XY1B-SP	2DN22802	YES	RETURNED	NO
XY1B-SP	2DN22HT1	YES	HQ	NO
XY1B-SP	2DN22HT2	YES	HQ	NO
XC1B-SP	2CN229KA	YES	HQ	NO
XC1B-SP	2CN22AG6	YES	HQ	NO
XC1B-SP	2CN22BD3	YES	HQ	NO
XY1B-SP	2DN227P6	YES	RETURNED	NO
XC1B-SP	2CN228R5	YES	HQ	NO
XC1B-SP	2CN22959	YES	HQ	NO
XC1B-SP	2CN22AG1	YES	HQ	NO
XC1B-SP	2CN22AG4	YES	HQ	NO
XY1B-SP	2DN21R85	YES	RETURNED	NO
XC1B-SP	2CN229D6	YES	HQ	NO
XC1B-SP	2CN22CG7	YES	HQ	NO
XC1B-SP	2CN22CG6	YES	HQ	NO
XY1B-SP	2DN21WBA	YES	RETURNED	NO
XY1B-SP	2DN21XKA	YES	RETURNED	NO
XC1B-SP	2CN22CG1	YES	HQ	NO
XC1B-SP	2CN22A93	YES	HQ	NO
XY1B-SP	2DN22P51	YES	HQ	NO
XY1B-SP	2DN22QA9	YES	HQ	NO
XY1B-SP	2DN22KK3	YES	HQ	NO
XY1B-SP	2DN22KK5	YES	HQ	NO
XY1B-SP	2DN211Y2	YES	RETURNED	NO
XC1B-SP	2CN223C3	YES	HQ	NO
XC1B-SP	2CN223QA	YES	HQ	NO
XC1B-SP	2CN22409	YES	HQ	NO
XC1B-SP	2CN22411	YES	HQ	NO
XC1B-SP	2CN22445	YES	HQ	NO
XY1B-SP	2DN215C2	YES	RETURNED	NO
XY1B-SP	2DN215F7	YES	RETURNED	NO
XC1B-SP	2CN226G9	YES	HQ	NO
XY1B-SP	2DN21HT7	YES	RETURNED	NO
XY1B-SP	2DN21R88	YES	RETURNED	NO
XY1B-SP	2DN21R97	YES	RETURNED	NO
XY1B-SP	2DN21WB9	YES	RETURNED	NO

List of impacted Serial Numbers (as provided by Hoya)		Distributor Response		
Model #	Serial #	Product Received? Yes/No	Location of Product [Used/at warehouse/ name of customer]	PSN label attached to product? Yes/No [NA in case product has been used]
XY1B-SP	2DN21XK9	YES	RETURNED	NO
XY1B-SP	2DN22856	YES	HQ	NO
XC1B-SP	2CN227Z2	YES	HQ	NO
XC1B-SP	2CN228X6	YES	HQ	NO
XC1B-SP	2CN22AG8	YES	HQ	NO
XC1B-SP	2CN22BD7	YES	HQ	NO
XC1B-SP	2CN22152	YES	HQ	NO
XC1B-SP	2CN22171	YES	HQ	NO
XC1B-SP	2CN223D8	YES	HQ	NO
XC1B-SP	2CN223N8	YES	HQ	NO
XC1B-SP	2CN22415	YES	HQ	NO
XC1B-SP	2CN22428	YES	HQ	NO
XC1B-SP	2CN22441	YES	HQ	NO
XY1B-SP	2DN215K1	YES	RETURNED	NO
XC1B-SP	2CN224R7	YES	HQ	NO
XC1B-SP	2CN226J3	YES	HQ	NO
XY1B-SP	2DN218P3	YES	HQ	NO
XY1B-SP	2DN21HU8	YES	RETURNED	NO
XY1B-SP	2DN21VKA	YES	RETURNED	NO
XY1B-SP	2DN225T9	YES	HQ	NO
XY1B-SP	2DN22889	YES	RETURNED	NO
XY1B-SP	2DN22B07	YES	RETURNED	NO
XY1B-SP	2DN22B08	YES	RETURNED	NO
XC1B-SP	2CN22946	YES	HQ	NO

Section 2


Reconciliation is complete: **YES/NO**

Location of all impacted products has been identified: **YES/NO**

Quantity not possible to reconcile: N/A

Reason(s): N/A

Section 3

Role/Title	Name	Signature	Date
GENERAL MANAGER	ONG AIK SEIONG		12/7/2023

Important Note: Please return completed form by email to [insert name/title/email address] **within 15 business days.**

We, Medfray Medical does not perform any FSN activity and would like to return all impacted product back to HOYA.

